

VETERINARY FLUID WARMER

TEMPCARE

New in
Canada

The Tempcare has been designed to warm I.V. fluids near to the body temperature of the small animal patient. This is essential during surgical procedures where the patient experiences body heat loss.



Life-threatening hypothermia occurs commonly with shock, trauma, adverse weather exposure and debility.

External body warming methods are often inadequate alone.

The Tempcare Veterinary Fluid Warmer can be a vital component to the care of the ill or compromised patient.

Advantages of Using Tempcare

- Tempcare is capable of warming all
- IV fluids available **including blood.**
- Tempcare is lightweight, portable
- and compact.
- **No special IV sets are required.**
- Tempcare is compatible to
- most regular sets.
- Fast and easy set-up.
- One touch on/off pre-set operation.
- Easy to clean and maintain.

Tempcare Warming Performance

- Heat is conducted from the groove of the heating plate to the fluid inside the tube of the IV set. The temperature of the heating plate is always controlled at 46°C (115°F) by sensors, so a constant safe fluid temperature flowing to the patient is maintained.
- Maximum temperature entering the animal is 40°C.

Safety Mechanisms

- Temperature sensors control the temperature of the heating plate and fluid to prevent overheating.
- A temperature-controlled fuse shuts off the power supply in the event that all of the sensors fail.

* *Water-resistant design*

BORDERLINK

Veterinary
Supplies Inc.

233 Mapleview Dr. E., Suite 2
Barrie, ON
L4N 0W5

Phone: 1-888-531-1511
Fax: 705-739-4542
Email: borderlink@backland.net
www.borderlink.ca

Available through Borderlink Veterinary Supplies